

OMRON

GentleTemp™

Ear Thermometer

Model MC-510

Ideal for children



All for Healthcare

Backlit Screen
For easy temperature taking at night

Compact & Lightweight
Ease for travel and storage


*Fast One-second
Measurement*

*Automatic
Power Off*

Taking temperatures has never been easier with OMRON Gentle Temp™ Ear Thermometer. Backlit screen enables temperature taking at night without disturbing your sleeping child.

Fast One-Second Mode

Recommended for self-measurement or when you are experienced in taking temperature from the ear

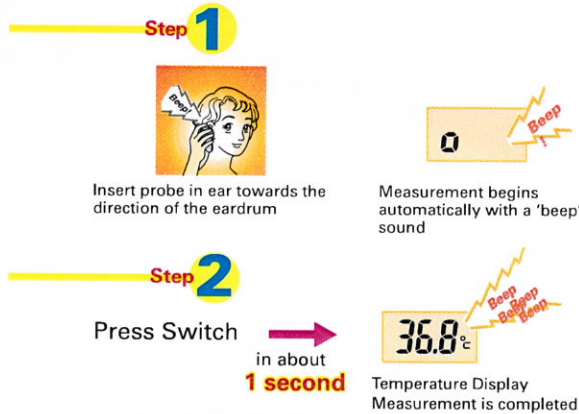

For Backlighting

Press the switch again after measurement to illuminate the backlit screen for 2 seconds. You can easily read the results.

ADVICE FOR MEASURING THE TEMPERATURE OF INFANT AND YOUNG CHILDREN



When infant is lying down



When infant sits up



Measuring for a child



If the ear is too small to insert the probe: While slightly pulling the ear towards the back, insert probe without force so that it just cover the external auditory canal.

Specifications:

Model	MC-510
Name	Ear Thermometer
Voltage	3V DC (lithium battery CR2032)
Power consumption	0.05 W
Battery life	Measurements of approx. 2000 times (when used consecutively under the environment of 23°C or 73.4°F, indicates that measurement can be carried out up to a certain number of times until the voltage of the battery has dropped to the low battery stage while the backlight keeps illuminating)
Sensor	Thermopile
Temperature indication	In 3 digits in +°C in the unit of 0.1°C (or in 4 digits in +°F in the unit of 0.1°F)
Measurement range	34 to 42.2°C (93.2 to 108°F)
Weight	Approx. 50g (including the battery)
External dimensions	Approx. 45 (W) x 94 (H) x 58 (D)
Operating environment	Ambient temperature: 10 to 40°C (50 to 104°F) Relative humidity: 30 to 85%
Accessories	Storage case, Lithium battery CR2032, Instruction manual, *probe covers (10 pieces) A probe cover is already attached to the main unit
Options	Probe cover exclusive for Gentle Temp™ series

Omron reserves the right to change specifications without prior notice.

*Omron is not liable for use of probe cover, unless it is designed and supplied by Omron.

OMRON HEALTHCARE Co.,Ltd
53, Kunotsubo, Terado-cho,
Muko, Kyoto 617-0002 Japan

Asia Pacific HQ
OMRON HEALTHCARE SINGAPORE PTE LTD
438A Alexandra Road, #05-05/08,
Alexandra Technopark, Singapore 119967
www.omron-healthcare.com.sg

Distributed by:



OMRON HEALTHCARE EUROPE B.V.
Scorpius 33, 2132 LR Hoofddorp
THE NETHERLANDS

CE0197